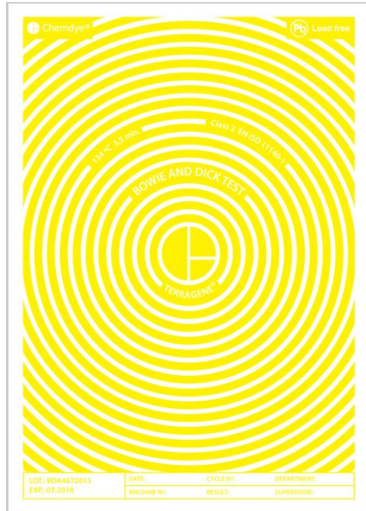
TERRAGENE

Product Name

BDA4 - CHEMDYE® TERRAGENE® Bowie & Dick Test Sheet Chemical Indicator.

Usage

Bowie & Dick test to evaluate air removal and steam penetration in vacuum-assisted Steam sterilizers. Cycles of 3.5 min at 134 °C.

Applicable regulation

ISO 11140-1:2014, ISO 11140-3:2007 and ISO 11140-5:2009

Product classification

Class 1, according to the risk.

Authorization

Designed under Quality Systems EN ISO 9001:2008 and EN ISO 13485:2012.

Product characteristics

Type 2 Chemical Indicator.

A4 size sheets, 80 g/m².

Initial color: **yellow**

Final color: **dark brown/ black**

NOTE:

Air removal or steam quality problems will be shown with a heterogeneous color change into dark brown/black on the test sheet, such as lighter areas in the center.

100 % Lead Free.

Environmental conditions during manufacture

15-30 °C (room temperature), RH 30-60 %.

Storage conditions

T = 10-30 °C, RH 30-80 %, keep out of direct light.

Transportation conditions

Storage conditions should be strictly followed. Transport in closed and reinforced boxes in order to avoid damages. Transportation of this product does not represent a risk for health.

Shelf-life

5 years

Packing

Envelopes of 50 sheets.

Packing information: product code and description, presentation, process for intended use, classification according to regulation, manufacturer information and data on label.

Labelling

Only on product's box: batch number, manufacture and expiration date.

Possible target markets

Healthcare, Food, Pharmaceutical and medical Industries.

Other important information (optional)

Read product's directions for use thoroughly before use.

Note: When necessary, limits for each product's parameters should be specified.

TERRAGENE

Description of identified and/or predictable risks which could represent a risk for the product.

Do not store the product near sterilizing agents.

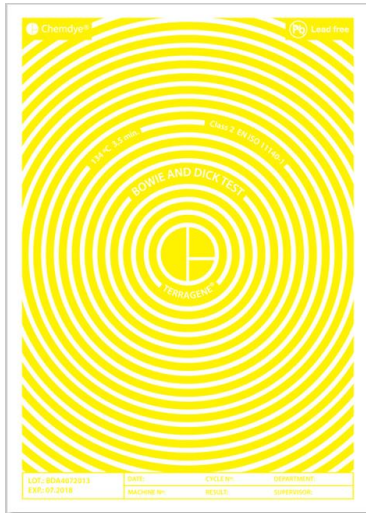
Do not expose this product to EO, Dry Heat or any sterilization processes other than Steam.

Do not use this product for air removal monitoring of gravity displacement Steam, Dry Heat, EO, or other low temperature sterilization processes.

Do not reuse the sterilizer until the whole sheet has changed homogeneously into the color indicated in the product.

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